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**INTENDED USE**

The bath stool and the bathboard are designed to allow disabled people to be able to enjoy a bath while sitting on them, reducing muscle fatigue due to standing position as well as slip and injuries risk.

The bath stool can be used inside or outside of the tube or in the shower, The board can be adjusted & fixed on the edges of the bathtub.

**FEATURES:**

- Adjustable in height for the code 0807545.
- Adjustable mounts for the code 0808366, to fit better in the width of your bathtub.

**DIMENSIONS:**

REF.: 0808366					
	69,5cm	26cm	41-62cm	-	100Kg
REF.: 0807545					
	Φ32cm	35-53cm	100kg		

**ASSEMBLING:**

The product is, assembled ready to use. You may need to assemble part of the metal legs.

**FURTHER INSTRUCTIONS:**

- Position the on the floor and check that it is correctly in this position.
- If necessary, adjust the height.
- Transfer the user in the chair.
- Make sure that the chair is secure and does not swing before starting the bath.
- Do not leave the user alone, unsupervised.

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**BEFORE EVERY USE:**

- Check, if the product is properly assembled and all screws are secure and well screwed on.
- That the legs end firmly touching the floor or on the bathtub.
- Check if the frame is damaged in order to guarantee a safe use of the product. (There are no cracks or fractures in the frame).
- In case of damage do not use the product and contact your dealer for further instructions.
- We do not recommend that the product is issued to anyone without first carrying out a full assessment of their specific needs. It is important that the user is given the necessary support by the product as prescribed by a qualified professional.

**GENERAL SAFETY WARNINGS:**

- This manual MUST be given to the user of the product.
- If you don't read this user manual, it is preferable not to use this product or another available part. If you don't understand the usage or the precautions, please contact with the dealer or the proper technical person before you use the product because may be caused damage.
- Keep away the product from heat sources.
- READ & Save this document for future reference.
- Be careful when children are nearby and do not allow the children to play with the product.
- Special care is always required where there are moving parts that can cause limb entrapment and injury.
- Do not attempt to lift the product by any removable parts.
- Do not use the product seat for a different purpose from the intended one.
- Do not exceed the weight limit of the safe working load.

**DECLARATION OF CONFORMITY:**

We are solely responsible for declaring that the Medical Devices mentioned in this statement are of Low-Risk Class (Class I) and comply with the requirements of the European Regulation 745/2017 and where appropriate, the standards and legislation referred to.

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**ENVIRONMENTAL PROTECTION:**

If one day you find that your product needs to be replaced or it is no longer working for you, consider protecting the environment:

- 1) Do not dispose your product along with the rest of the public waste (this is also the meaning of the shown recycling sign).
- 2) Contact your Public Authorities and they will instruct you of the Recycling centers to which your product must be disposed.
- 3) Correct disposal of your product helps the protection of the environment as well as the recycling of the product's components.

**CAUTION (!):**

- In case of damage or malfunction of your product DOT NOT use the product and please contact the authorized and trained representative.
- Do not use the product for a purpose not indicated in this manual
- MOBIAK S.A declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product.
- MOBIAK S.A reserves the right to change the information contained in this document without previous notice
- There is a  $\pm 3\%$ . Deviation for the Dimensions of the Products.

**MAINTENANCE, CLEANING & DISINFECTION INFORMATION:**

- Periodic checking and inspection of the product will help to maintain its life and effectiveness.
- Clean each section of the product frequently.
- Remove the evident dirt before to proceed with the most accurate cleaning.
- Use only a cloth and mild soap on surfaces that need cleaning.
- Then use a clean cloth and dry well before using the product.
- DO NOT leave the product wet.
- If you need to disinfect the product, use a common, mild disinfectant by wiping down all generally accessible surfaces
- Do not use an abrasive detergent cleaner, Never use acids.
- Protect the item from scratches, cuts and punctures.

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**INCIDENT REPORTING:** Any serious incident that occurs in relation to use of the device must be reported to the manufacturer and a competent authority of the Member State in which the user and / or patient resides.

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**WARRANTY:**

The product is guaranteed for a period of one (1) year from the purchasing date. The warranty apply on factory defects & DOES NOT apply on damages resulting from misuse, poor maintenance, modification, overuse, or non-compliance with the product's operating instructions. Also DOES NOT apply on the parts, that can wear out during use or over time. Damages or defects caused by natural disasters, unauthorized maintenance or repair, power supply problems (where applicable) & carriers are not covered by the warranty. No service or spare part is applied unless it has been inspected by an authorized service department or the manufacturer of the product. Shipping costs for goods of warranty are to be paid by the end user- customer or by the dealer.

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USER MANUAL: ROUND BATH STOOL & ADJUSTABLE BATHTUB BENCH

REF.: 0807545

REF.: 0808366

MOBIAK S.A  
ΚΑΘΙΑΝΑ ΑΚΡΩΤΗΡΙΟΥ, ΧΑΝΙΑ - ΚΡΗΤΗ  
KATHIANA AKROTIRIOU CHANIA - CRETE  
T: +30 28210 63222 | WEB: www.mobiakcare.com